IT

Descrizione

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Información:

Further technical information is available from the following address:

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This mold is customized and must be used in the ear canal to protect against dangerous noise levels.

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Laboratorios Sonora S.L hereby declares that the PPE (EPI) DM complies with Regulation (EU) 2016/425. The full proof of the EU Declaration of Conformity is available at the following internet address:
www.lolacustica.com

Stand der Information / Date of information / État de l'information / Versión de l'información / Stato delle informazioni / Laatste wijziging: 12.2022

Technical data in accordance with EN 352-2:2020
Datos técnicos conforme a EN 352-2:2020

H-high frequency attenuation (f > 2000 Hz); M-medium frequency attenuation (f > 500 Hz and f < 2000 Hz); L-low frequency attenuation (f < 500 Hz) SNR-Single Number Rating

Frequency	63	125	250	500	1000	2000	4000	8000		SNR	H	M	L
DLO® Silicone with DM filter, black													
Average attenuation value Mean (dB)	25	26.7	27.4	26.9	26.3	28.3	20.2	26.7	Mean [dB]	24.5	23.1	24.6	25.6
Standard deviation STD dev (dB)	3.5	3.4	3.6	3.7	3.2	3.8	2.6	2.8	STD [dB]	1.5	2	1.5	1.9
Assumed protection effect APV (dB)	21.5	23.3	23.8	23.2	23.1	24.5	17.6	23.9	Value [dB]	23	21	23	24

										SNR	H	M	L
DLO® Silicone with DM filter, white													
Average attenuation value Mean (dB)	23.7	24.3	22.5	2.7	21.9	18.8	19.9	16.6	Mean [dB]	19.7	17.9	20	20.9
Standard deviation STD dev (dB)	4.2	3.8	2.3	2.2	2.5	4.1	2.5	2.9	STD [dB]	1.7	2.2	2	1.7
Assumed protection effect APV (dB)	19.5	20.5	20.2	19.3	19.4	14.7	17.4	13.7	Value [dB]	18	16	18	19

										SNR	H	M	L
DLO® Silicone with DM filter, blue													
Average attenuation value Mean (dB)	18.8	18.4	17.7	15.9	18.1	18.3	19.4	17.5	Mean [dB]	18.2	17.6	17	16.8
Standard deviation STD dev (dB)	3.9	2.9	2.9	3.3	4	4.4	2.5	4.5	STD [dB]	2.1	2.6	2.3	2.4
Assumed protection effect APV (dB)	14.9	15.5	14.8	12.6	14.1	13.9	16.9	13	Value [dB]	16	15	15	14

Dreve

Protection & Comfort

DLO® Music

Sound Technology by
DYNAMIC EAR COMPANY

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EN

Description

You are now in possession of an individual hearing protection earmold which ought to have a life of at least 2 years. Please read the instructions carefully before using the earmold.

DLO®

As personal protective equipment (PPE) the Dreve hearing protection earmolds including DM filters are type approved according to Regulation (EU) 2016 / 425. They meet the essential health and safety requirements in the workplace. The Notified Body for approval and monitoring of DLO® is PZT GmbH (1974) in D-26389 Wilhelmshaven.

Insertion

Take the earmold out of the packing with clean hands. For the correct insertion touch the noise plug with thumb and index finger and insert the earmold carefully with a slightly backwards turning movement. In order to ease the insertion pull the ear with one hand to the back/upwards and insert the ear-mold with the other hand.

In order to distinguish the two earmolds they have been marked with different colors (right = red / left = blue). The earmold can only attain complete protection, if you insert it carefully and wear constantly during exposing to noise. Even if you remove the earmold for a short moment, this may damage your hearing. We would recommend you to have your earmold checked annually by your authorized hearing healthcare professional.

Removal

If present grab the earmold at the bridge and remove it carefully out of the ear.

You generally need to avoid pulling and dragging the cord.

Cleaning

Your earmold has been designed for a use of several years. Therefore it has to be cleaned carefully after each use with a moisture cleaning tissue. For thorough cleaning and disinfection we recommend our **OtoVita®** care range. Avoid contact with aggressive solvents (e. g. acetone)! For cleaning, keep the filter system with the thumb closed while cleaning the ear facing surface with the help of an **OtoVita®** cleaning tissue. The filter system has to be kept away from moisture, otherwise the attenuation effect is impaired. Store the earmold in the original packing.

Important advice!

The earmold might suffer from certain chemical substances. Further indications may be requested from the manufacturer. In case these recommendations are not followed the protective action of the earmold might be affected considerably. We do therefore recommend to insert and care the earmold according to our indications. The earmold is available with a cord. "Warning: Do not wear these hearing protection earmolds if there is a risk of the cord being torn away or cord entanglement." Sudden or rapid removal of hearing protection may cause damage to the eardrum

Technical data in accordance with EN 352-2:2020

Enjoy your personal hearing protection!

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El organismo notificado para la aprobación y vigilancia de DLO® es PZT GmbH (1974) en D-26389 Wilhelmshaven.