IT

Descrizione

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(EN)

Description

We congratulate you on the purchase of this custom-made hearing protection earmold to protect your hearing from harmful noise. The lifetime is intended for at least 2 years. Please read the instructions carefully before using the earmold.

DLO*

As personal protective equipment (PPE) the Dreve hearing protection earmolds including DI filters are type approved according to Regulation (EU) 2016 / 425. They meet the essential health and safety requirements in the workplace. The DLO* with DI filter has been approved by PZT GmbH (1974) in D-26389 Wilhelmshaven.

Insertion

Take the earmold out of the packing with clean hands. For the correct insertion touch the noise plug with thumb and index finger and insert the earmold carefully with a slightly backwards turning movement. In order to ease the insertion pull the ear with one hand to the back/upwards and insert the earmold with the other hand.

In order to distinguish the two earmolds they have been marked with different colors (right = red / left = blue).

The earmold can only attain complete protection, if you insert it carefully and wear constantly during exposing to noise. Even if you remove the earmold for a short moment, this may damage your hearing. We would recommend you to have your earmold checked annually by your authorized hearing healthcare professional. According to § 8 of the LärmVibrationsArbSchV (German regulation for occupational protection from noise and vibrations), a recurring function check is required at least every two years.

Removal

If present grab the earmold at the bridge and remove it carefully out of the ear. You generally need to avoid pulling and dragging the cord.

Cleaning

Your earmold has been designed for a use of several years. Therefore it has to be cleaned carefully after each use with a moisture cleaning tissue. For thorough cleaning and disinfection we recommend our **OtoVita*** care range. Avoid contact with aggressive solvents (e. g. acetone)! For cleaning, keep the filter system with the thumb closed while cleaning the ear facing surface with the help of an **OtoVita*** cleaning tissue. The filter system has to be kept away from moisture, otherwise the attenuation effect is impaired. Protect the earmolds from extreme temperatures and store them in the packaging provided.

Important advice!

The earmold might suffer from certain chemical substances. Further indications may be requested from the manufacturer. In case these recommendations are not followed the protective action of the earmold might be affected considerably. We do therefore recommend to insert and care the earmold according to our indications. The earmold is available with a cord. "Warning: Do not wear these hearing protection earmolds if there is a risk of the cord being torn away or cord entanglement." Undesirable side effects are not to be expected when wearing this PPE. However, immune reactions (e. g. allergies) or local discomfort (e. g. irritations in the auditory canal) cannot be ruled out in principle. If you become aware of undesired side effects, please let us know in any case.

Technical data in accordance with EN 352-2:2002

Dreve hereby declares that the PPE DLO* with DI filter corresponds to Regulation (EU) 2016 / 425. The full text of the EU Declaration of Conformity is available at the following Internet address: smartorder.dreve.de

Enjoy your personal hearing protection!

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